

# Needlestick prevention that meets your needs



- ▶ minimizes exposure to the contaminated needle
- ▶ allows for one-handed activation
- ▶ integrated safety mechanism
- ▶ once activated, the needle is safely retracted through disposal
- ▶ easy to use

**VANISHPOINT® PORTED IV CATHETERS ARE SAFE AND EFFECTIVE**

VanishPoint® IV Catheters-Ported 50 per box/200 per case

Catalog Number	Gauge	Length Inch	mm	Catheter OD	Catheter Material	Color Code
38221	24	¾	19	.07mm	Radiopaque	Yellow
38331	22	1	25	.09mm	Radiopaque	Blue
38441	20	1¼	32	1.1mm	Radiopaque	Pink
38541	18	1¼	32	1.3mm	Radiopaque	Green
38641	16	1¼	32	1.7mm	Radiopaque	Grey
38741	14	1¼	32	2.1mm	Radiopaque	Orange

Additional sizes available. For a complete list, please contact RTI or visit our website at [www.vanishpoint.com](http://www.vanishpoint.com).

## Other Available Safety Products

**NEW!**



VanishPoint® Blood Collection Set



VanishPoint® Blood Collection Tube Holder



VanishPoint® Syringe



Patient Safe® Syringe



© 2009, 2013 Retractable Technologies, Inc.  
All rights reserved. M030 REV2 EN, AUG 2013

511 Lobo Lane  
Little Elm, Texas 75068-0009 USA  
Tel.: +1 972-294-1770  
Fax: +1 972-294-4400  
[rti.intl@vanishpoint.com](mailto:rti.intl@vanishpoint.com)

**UNCOMPROMISING SAFETY FOR CLINICIANS AND PATIENTS**



**VANISHpoint®**  
**IV CATHETER-PORTED**

**The New Standard for Safety®**

1

Perform venipuncture, observing for blood return. Advance the catheter slightly to assure full entry into the vein.



2

Holding the flashback chamber stationary, disengage the hub from introducer needle and advance the catheter into the vein.




3

Before withdrawing the introducer needle from the catheter, depress the color coded end of the needle housing to activate the automated retraction.



4

The introducer needle will retract into the housing. Dispose of the introducer needle in an appropriate sharps container.

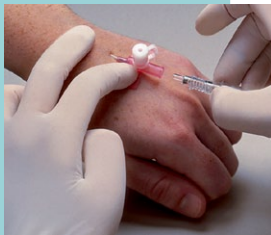

To inject medication using the injection port, use aseptic technique to:

Remove the medication cap. Securely attach the luer end of the syringe to the injection port and inject medication.

After injection, immediately close the injection portcap.



# the VANISHPOINT<sup>®</sup> ADVANTAGE

A safety IV catheter with injection port  
featuring the patented VanishPoint<sup>®</sup>  
automated retraction



*VanishPoint<sup>®</sup> ported IV catheters are available in a variety of lengths and gauges*

Capped injection port allows for safe and easy medication injection

Clear finger grips and flashback chamber allow for handling ease and flashback visualization



Sharp introducer needle for insertion ease and patient comfort

Wings provide stabilization and hub securement

Catheter hub, wings and housing are color coded according to ISO standards for easy identification of catheter gauge



Automatic needle retraction occurs with slight depression of the needle housing. Once activated, the needle is safely retracted into the housing, reducing the risk of needlestick injuries