



## Needlestick prevention that **meets your needs**

- ▶ Virtually eliminates exposure
- ▶ Single-handed, passive activation
- ▶ Hands remain behind the sharp
- ▶ Integrated safety mechanism
- ▶ Prevents recapping and needle removal
- ▶ Remains safe through disposal
- ▶ Easy to use, requires minimal training
- ▶ Safe and effective for patients

Catalog No.	Description	
<b>VanishPoint® U-100 insulin syringes</b> 100 per box/800 per case		
15391	0.5mL VanishPoint insulin syringe (half-unit)	30G x 3/16" <b>NEW</b>
15371	0.5mL VanishPoint insulin syringe (half-unit)	30G x 5/16" <b>NEW</b>
15291	0.5mL VanishPoint insulin syringe	30G x 3/16"
15271	0.5mL VanishPoint insulin syringe	30G x 5/16"
15221	0.5mL VanishPoint insulin syringe	30G x 1/2"
10291	1mL VanishPoint insulin syringe	30G x 3/16"
10271	1mL VanishPoint insulin syringe	30G x 5/16"
10281	1mL VanishPoint insulin syringe	29G x 5/16"
10211	1mL VanishPoint insulin syringe	29G x 1/2"

<b>1mL VanishPoint® syringes</b> 100 per box/800 per case		
10131	1mL VanishPoint tuberculin syringe	27G x 1/2"
10151	1mL VanishPoint tuberculin syringe	25G x 5/8"
10161	1mL VanishPoint syringe	25G x 1"

<b>VanishPoint® Allergy Syringe Tray</b> 25 per tray/1000 per case		
10134	1mL VanishPoint allergy syringes	27G x 1/2"

<b>3mL VanishPoint® syringes</b> 100 per box/600 per case		
13021	3mL VanishPoint syringe	27G x 1 1/2"
10301	3mL VanishPoint syringe	25G x 5/8"
10391	3mL VanishPoint syringe	25G x 1"
13011	3mL VanishPoint syringe	25G x 1 1/2"
10311	3mL VanishPoint syringe	23G x 1"
10321	3mL VanishPoint syringe	23G x 1 1/2"
10331	3mL VanishPoint syringe	22G x 1"
10341	3mL VanishPoint syringe	22G x 1 1/2"
10351	3mL VanishPoint syringe	21G x 1"
10361	3mL VanishPoint syringe	21G x 1 1/2"
10371	3mL VanishPoint syringe	20G x 1"
10381	3mL VanishPoint syringe	20G x 1 1/2"

<b>5mL VanishPoint® syringes</b> 100 per box/600 per case		
<b>10mL VanishPoint® syringes</b> 100 per box/600 per case		

5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at [www.retractable.com](http://www.retractable.com).



511 Lobo Lane  
Little Elm, Texas 75068-5295 USA  
Tel.: (972) 294-1770  
Fax: (972) 294-4400  
Toll Free: 1-888-703-1010  
rtisales@retractable.com

©2008, 2022 Retractable Technologies, Inc.  
All rights reserved. M005 REV.16 EN, DEC 2022



The new standard for safety®



Effectively reduces the risk of needlestick injuries

A study comparing the effectiveness of passive SED (safety-engineered devices) with that of active SED, concluded that passive SED (such as the VanishPoint® syringe) significantly reduced the needlestick injury rate.<sup>1</sup>

### Attached needle design:

- Maintains a closed system
- Prevents leakage and contamination
- Reduces deadspace
- Prevents unnecessary needle changes

"Changing needles between drawing vaccine from a vial and injecting it into a recipient is unnecessary."<sup>2</sup>

<sup>1</sup>A. J. Goris, et al., "Reducing Needlestick Injuries from Active Safety Devices: A Passive Safety-Engineered Device Trial," *AOHP Journal*, vol. 34, no. 2 (Spring 2014), pp. 14-18.  
<sup>2</sup>CDC (2002), General recommendations on immunization: recommendations of the Advisory Committee on Immunization Practices (ACIP) and the American Academy of Family Physicians (AAFP), *MMWR* 51(RR02):1-36.

**Safe** for you and your patients



**VANISHPOINT®**  
Safety by design


1

Use standard procedure for drawing up medication.



2

Prepare and give injection using aseptic technique according to institutional policy.

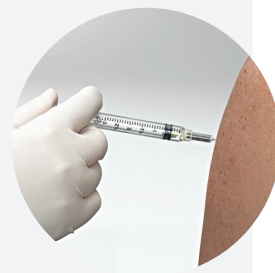


3

While the needle is still in the patient, fully depress the plunger to activate retraction.



4

Dispose of VanishPoint® syringe in an appropriate sharps container.

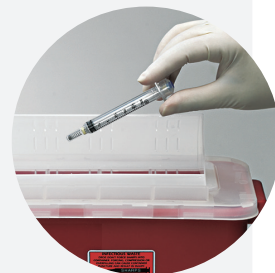


# THE VANISHPOINT® ADVANTAGE

## Effective safety with automated retraction technology


Clear, unobstructed calibrations allow for accurate dosing

Triple beveled, lubricated needle provides patient comfort

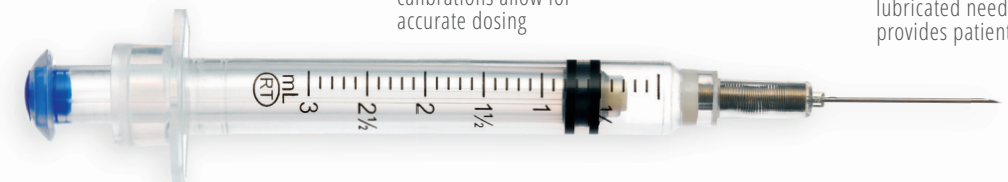

**EASY, ONE-HANDED ACTIVATION**

**NO CONTAMINATED SHARP EXPOSED**

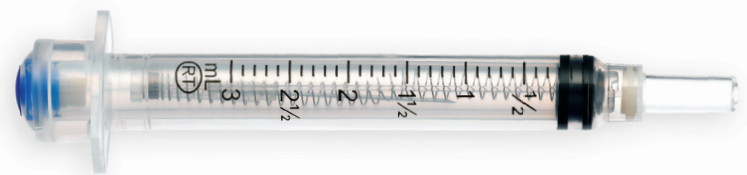

Needle retracts directly from patient into syringe

Pre-removal activation prevents exposure to contaminated sharp



VanishPoint® insulin syringes **NOW AVAILABLE WITH**

- 5mm and 8mm needles
- Half-unit increments for accurate small dose delivery




Activated VanishPoint® syringes require less disposal space than other safety syringes and prevent disposal-related injuries.


### Other available safety products



EasyPoint® Needles

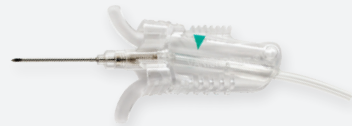

VanishPoint® Blood Collection Set

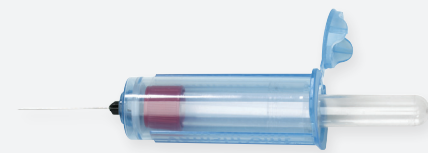

VanishPoint® Blood Collection Tube Holder



VanishPoint® IV Catheter

