

# EASYPPOINT®

retractable needle

- AUTOMATED NEEDLE RETRACTION
- EFFECTIVELY REDUCES THE RISK OF NEEDLESTICK INJURIES
- EASY TO USE - REQUIRES MINIMAL TRAINING
- ACTIVATION CAN BE ACHIEVED IN A VARIETY OF WAYS
- MAY BE ACTIVATED BEFORE OR AFTER REMOVAL FROM PATIENT
- PRE-REMOVAL ACTIVATION REDUCES THE RISK OF EXPOSURE TO THE CONTAMINATED SHARP
- ACTIVATION OF SAFETY MECHANISM AUTOMATICALLY COVERS SYRINGE LUER TIP, REDUCING RISK OF EXPOSURE TO MEDICATION, BLOOD/BODY FLUIDS
- REMAINS SAFE THROUGH DISPOSAL

compatible with both luer lock & luer slip syringes



may be used with pre-filled syringes



## EASYPPOINT®

RETRACTABLE NEEDLES  
(50 per box/400 per case)



### Needles only

#### Catalog # Needle Size

##### For syringes 3mL and smaller

82211	30G x 1/2"
82091	25G x 5/8"
82011	25G x 1"
82031	23G x 1"

##### For syringes 10mL and smaller

85291	25G x 5/8"
85211	25G x 1"
85011	25G x 1 1/2"
85231	23G x 1"
85021	22G x 1"
85031	22G x 1 1/2"
85041	21G x 1"
85051	21G x 1 1/2"
85061	20G x 1"
85071	20G x 1 1/2"
85081	18G x 1"
85091	18G x 1 1/2"



### With 3mL syringes

#### Catalog # Needle Size

82391	25G x 5/8"
82311	25G x 1"
85311	25G x 1 1/2"
82331	23G x 1"
85321	22G x 1"
85331	22G x 1 1/2"
85341	21G x 1"
85351	21G x 1 1/2"
85361	20G x 1"
85371	20G x 1 1/2"
85381	18G x 1"
85391	18G x 1 1/2"

### OTHER AVAILABLE SAFETY PRODUCTS

VanishPoint® Syringe  
VanishPoint® Blood Collection Tube Holder  
VanishPoint® Blood Collection Set  
VanishPoint® IV Catheter  
Patient Safe® Syringe



RETRACTABLE  
TECHNOLOGIES, INC.  
www.retractable.com

511 Lobo Lane, Little Elm, TX 75068-5295  
P 972-294-1770 • TF 1-888-703-1010 • F 972-294-4400  
rtsales@retractable.com

©2015, 2021 Copyright Retractable Technologies, Inc. All rights reserved.

M098 REV.7 EN, JAN 2021

# EASYPPOINT®

retractable needle



— **EASY TO USE** —

- minimal change in injection technique
- allows for single-handed activation
- **clinician's hands and fingers remain behind the exposed sharp** at all times




Attach EasyPoint needle to any luer lock or luer slip syringe. Administer medication according to institutional policy.



Apply pressure to the color-coded tab and the safety chamber using a squeezing motion, allowing the needle to automatically retract into the safety chamber. **This device is designed for activation before or after removing the needle from the patient.**



Dispose in an appropriate sharps container per protocol of institution.

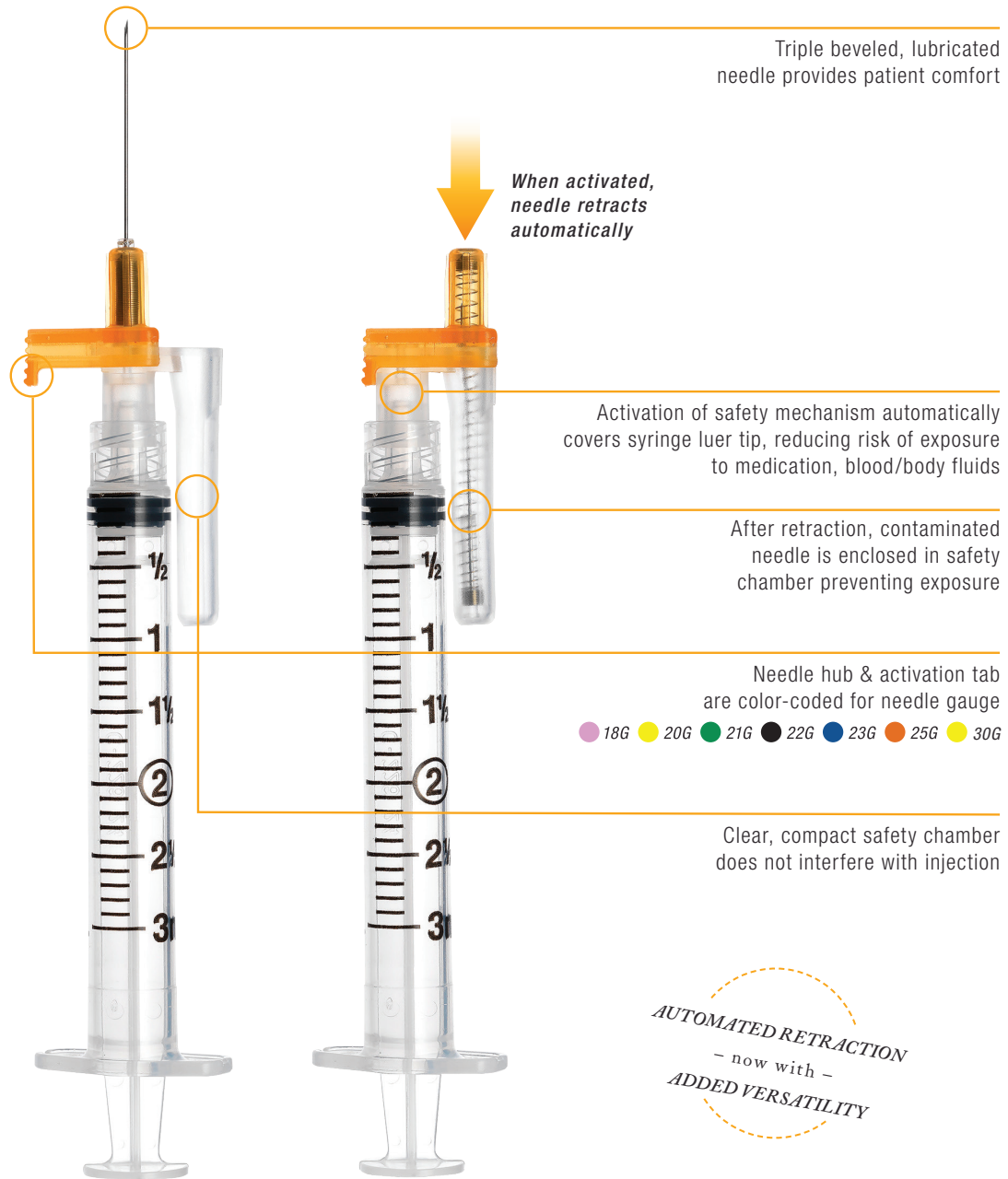

**EASYPOINT®**