

VanishPoint® Allergy Syringe Tray

Automated Retraction Technology

VanishPoint® Behälter für Allergie-Injektionsspritzen
Automatisierte Retraktionstechnologie

Plateau à seringues pour allergies VanishPoint®
Technologie de Rétraction Automatisée

Vassoio per siringhe per prove allergiche VanishPoint®
Tecnologia di Retrazione Automatizzata

Bandeja de jeringas para alergia VanishPoint®
Tecnología de Retracción Automatizada

VanishPoint® Set injectiespuiten tegen allergie
Geautomatiseerde Retractietechnologie

Bandeja de seringas de alergia VanishPoint®
Tecnologia de Retração Automatizada

VanishPoint® アレルギー注射器トレイ
自動引き込み技術

VanishPoint® Alerji Şırıngası Tepsisi
Otomatik Geri Çekme Teknolojisi

Taca strzykawk do alergii VanishPoint®
Technologia Automatycznego Wycofywania



Corporate Headquarters
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
USA
www.retractable.com

Sales and Marketing
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
USA
Fax: (972) 294-4400
Toll Free 1-888-703-1010

EC REP EU Auth. Rep.
(MDD93/42/EEC)
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

Manufacturing Location
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
USA

PATENTS—WWW.RETRACTABLE.COM / PATENTS




© 2000, 2020. Retractable Technologies, Inc. All rights reserved.

VanishPoint® Allergy Syringe Tray

Automated Retraction Technology

VanishPoint® Behälter für Allergie-Injektionsspritzen
Plateau à seringues pour allergies VanishPoint®
Vassoio per siringhe per prove allergiche VanishPoint®
Bandeja de jeringas para alergia VanishPoint®
VanishPoint® Set injectiespuiten tegen allergie

Bandeja de seringas de alergia VanishPoint®
VanishPoint® アレルギー注射器トレイ
VanishPoint® Alerji Şırıngası Tepsisi
Taca strzykawk do alergii VanishPoint®

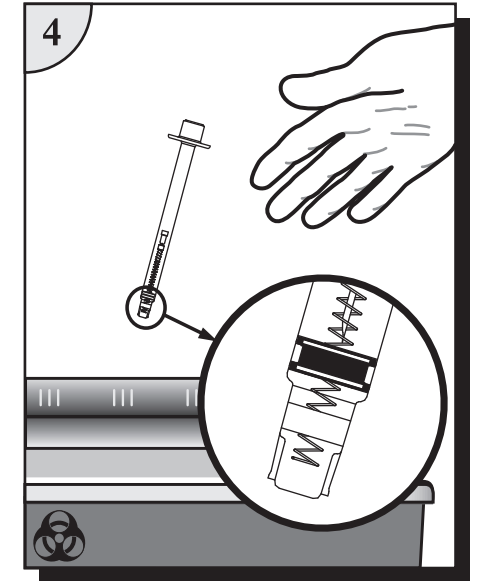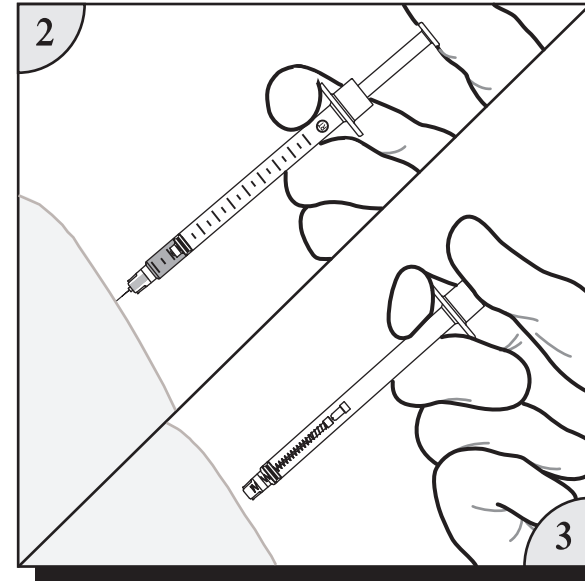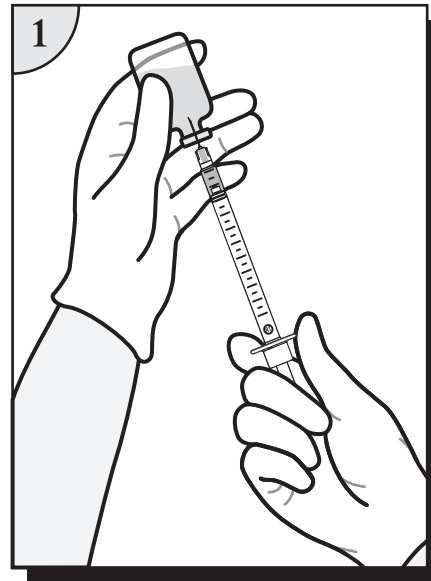

Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic.
- Contents are sterile until package is opened. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- For applications where full dose is not administered, expel remaining contents according to institutional policy and activate needle retraction.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

Instructions for Use:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. For injection into patients, continue depressing plunger to activate automatic needle retraction **while needle is still in patient. Full dose is administered only when needle retraction is activated.**
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint® syringe in an appropriate sharps container per protocol of institution.

