

## VanishPoint® II Allergy Syringe Tray

*Automated Retraction Technology*

VanishPoint® II Behälter für Allergie-Injektionsspritzen  
*Automatisierte Retraktionstechnologie*

Plateau à seringues pour allergies VanishPoint® II  
*Technologie de Rétraction Automatisée*

Vassoio per siringhe per prove allergiche VanishPoint® II  
*Tecnologia di Retrazione Automatizzata*

Bandeja de jeringas para alergia VanishPoint® II  
*Tecnología de Retracción Automatizada*

VanishPoint® II Set injectiespuiten tegen allergie  
*Geautomatiseerde Retractietechnologie*

Bandeja de seringas de alergia VanishPoint® II  
*Tecnologia de Retração Automatizada*

VanishPoint® II アレルギー注射器トレイ  
自動引き込み技術

VanishPoint® II Alerji Şıngası Tepsisi  
*Otomatik Geri Çekme Teknolojisi*

Taca strzykawk do alergii VanishPoint® II  
*Technologia Automatycznego Wycofywania*

### REAL SAFETY™



Corporate Headquarters  
Retractable Technologies, Inc.  
511 Lobo Lane, P.O. Box 9  
Little Elm, Texas 75068-0009  
USA  
[www.retractable.com](http://www.retractable.com)

Sales and Marketing  
Retractable Technologies, Inc.  
511 Lobo Lane, P.O. Box 9  
Little Elm, Texas 75068-0009  
USA  
Fax: (972) 294-4400  
Toll Free 1-888-703-1010

**EC REP** EU Auth. Rep.  
(MDD93/42/EEC)  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

Manufacturing Location  
Retractable Technologies, Inc.  
511 Lobo Lane, P.O. Box 9  
Little Elm, Texas 75068-0009  
USA

PATENTS--WWW.RETRACTABLE.COM / PATENTS




© 2022, 2023. Retractable Technologies, Inc. All rights reserved.

# VanishPoint® II Allergy Syringe Tray

*Automated Retraction Technology*

VanishPoint® II Behälter für Allergie-Injektionsspritzen  
Plateau à seringues pour allergies VanishPoint® II  
Vassoio per siringhe per prove allergiche VanishPoint® II  
Bandeja de jeringas para alergia VanishPoint® II  
VanishPoint® II Set injectiespuiten tegen allergie

Bandeja de seringas de alergia VanishPoint® II  
VanishPoint® II アレルギー注射器トレイ  
VanishPoint® II Alerji Şıngası Tepsisi  
Taca strzykawk do alergii VanishPoint® II

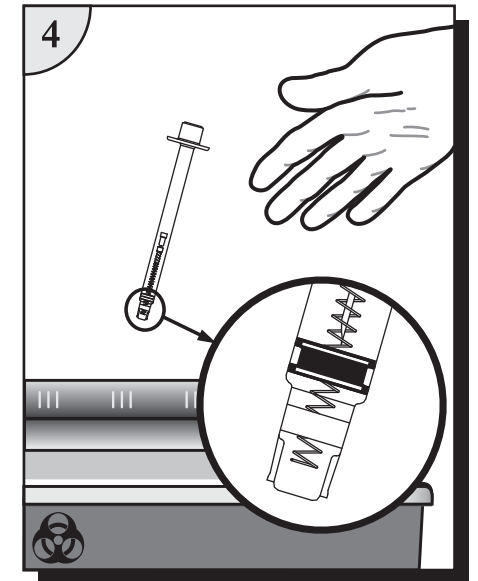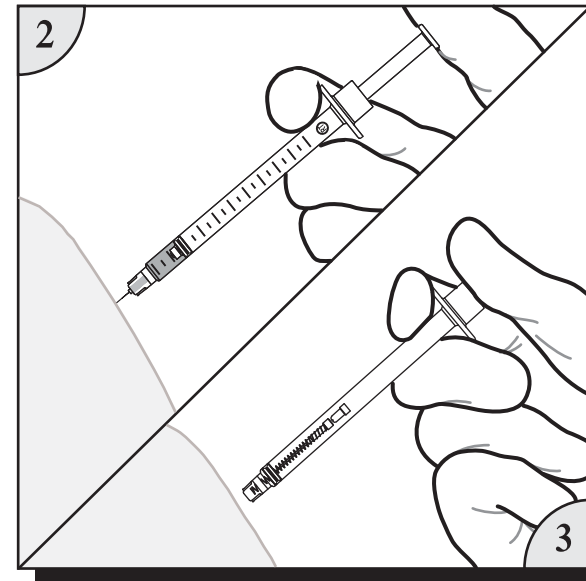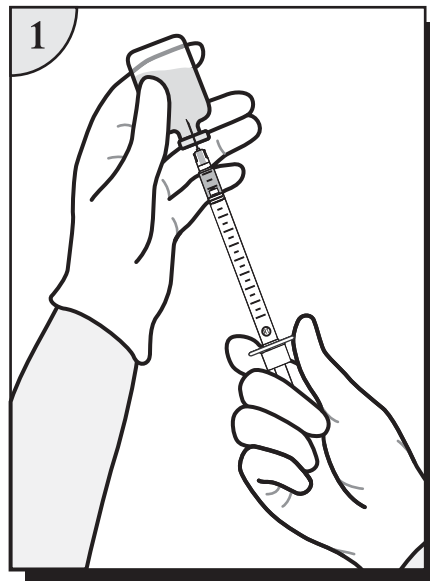

### Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic.
- Contents are sterile until package is opened. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- For applications where full dose is not administered, expel remaining contents according to institutional policy and activate needle retraction.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

### Instructions for Use:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. Continue depressing plunger to activate automatic needle retraction **while needle is still in patient. Full dose is administered when the plunger handle is fully depressed**, which activates the needle retraction mechanism.
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint® II syringe in an appropriate sharps container per protocol of institution.

