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


VanishPoint® II Auto Disable Safety Syringe

VanishPoint® II Sicherheitsspritze mit automatischer Sperre
Seringue de sécurité autorétractable VanishPoint® II
Siringa di sicurezza auto-disattivante VanishPoint® II
Jeringa de seguridad VanishPoint® II autodesechable
VanishPoint® II auto-uitschakelbare veiligheids- injectienaald

Seringa de segurança com autodesativação VanishPoint® II
VanishPoint® II オートディスエイブル安全シリンジ
VanishPoint® II Otomatik Kapanan Güvenlikli Şırınga
Bezpieczna samoniszcząca strzykawka VanishPoint® II

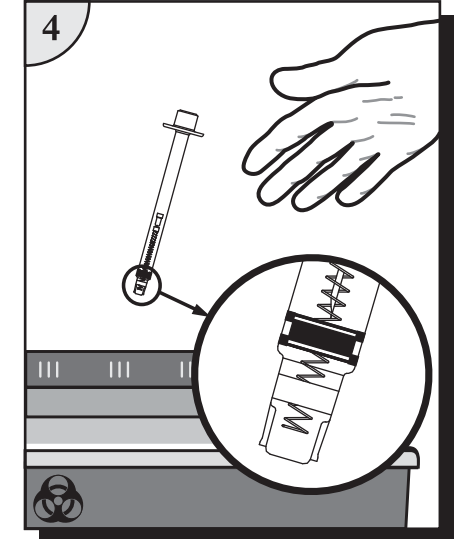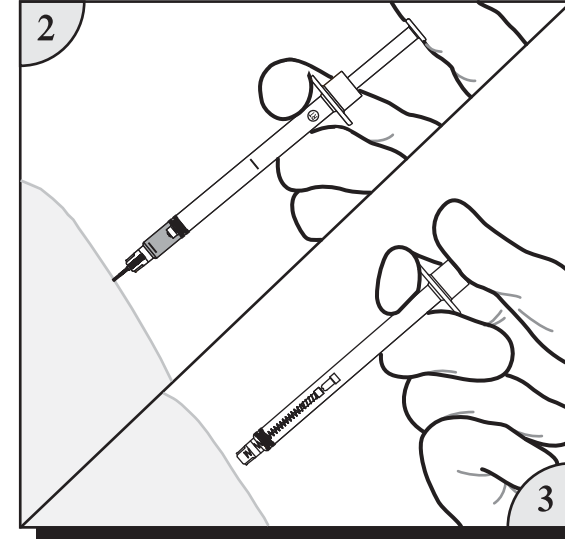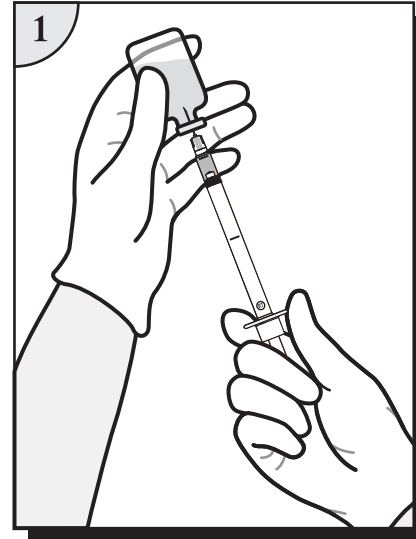REAL SAFETY™

RETRACTABLE®
TECHNOLOGIES, INC.
www.retractable.com

2
SINGLE USE

1
AUTO DISABLE

Corporate Headquarters
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
USA
www.retractable.com

Sales and Marketing
Retractable Technologies, Inc.
511 Lobo Lane, P.O. Box 9
Little Elm, Texas 75068-0009
Fax: (972) 294-4400
Toll Free 1-888-703-1010

EU Auth. Rep.
(MDD93/42/EEC)
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

Manufacturing Location
Jumin Bio-Technologies Co., Ltd.
1888 Huhang Road,
Jianghai Economic Zone
Fengxian, Shanghai
P. R. China

Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

Instructions for Use:

- Peel open the package where indicated.
- 1. Fill the syringe with medication and adjust plunger to the dose mark.
- 2. Continue depressing plunger to activate automatic needle retraction **while needle is still in patient. Full dose is administered when the plunger handle is fully depressed**, which activates the needle retraction mechanism.
- 3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container. Do not recap contaminated needles.
- 4. Dispose of VanishPoint® II syringe in an appropriate sharps container.

PATENTS--WWW.RETRACTABLE.COM/PATENTS

© 2005, 2022. Retractable Technologies, Inc. All rights reserved.

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EU Auth. Rep. Jumin Bio-Technologies Co., Ltd. 1888 Huhang Road, Jianghai Economic Zone Fengxian, Shanghai, P.R. China
MDSS GmbH Schiffgraben 41 D-30175 Hannover Germany